

Risk Assessment Patient Questionnaire

Before performing remote (at-home) administration of the capsule, providers must ensure the patient is not at risk for capsule aspiration and/or choking. During a telehealth visit, please review this questionnaire with the patient. **If the patient answers “YES” to any of the following questions, the patient should NOT receive remote administration of the capsule.** Additionally, to further assess if a patient may experience difficulties swallowing the capsule, you may choose to review current and past medicine that the patient has taken in a pill form to compare pill size to capsule size. The CapsoCam Plus capsule endoscope is 11mm (diameter) x 31mm (length).

- | | | |
|----------------------------------------------------------------------------------|------------------------------|-----------------------------|
| 1. Does the patient have any physical or cognitive disabilities? | <input type="checkbox"/> YES | <input type="checkbox"/> NO |
| 2. Is the patient younger than the age of 22 OR older than the age of 75? | <input type="checkbox"/> YES | <input type="checkbox"/> NO |
| 3. Does the patient have the following risk factors for aspiration? | | |
| • History of dysphagia (i.e. swallowing difficulties, especially large pills) | <input type="checkbox"/> YES | <input type="checkbox"/> NO |
| • Difficulties chewing food | <input type="checkbox"/> YES | <input type="checkbox"/> NO |
| • History of unexplained coughing, wheezing or choking (especially after eating) | <input type="checkbox"/> YES | <input type="checkbox"/> NO |
| • Lower level of consciousness | <input type="checkbox"/> YES | <input type="checkbox"/> NO |
| • Chronic respiratory conditions | <input type="checkbox"/> YES | <input type="checkbox"/> NO |
| 4. Has the patient been diagnosed with pneumonia IN THE PAST YEAR | <input type="checkbox"/> YES | <input type="checkbox"/> NO |

5. Has the patient EVER been diagnosed with aspiration pneumonia? YES NO
6. Does the patient have difficulty getting into sitting position independently? YES NO
7. Does the patient mix medicine with food or drinks to make it easier to swallow? YES NO
8. Does the patient break or crush tablets or open capsules to make it easier to swallow? YES NO
9. Does the patient have anxiety swallowing large vitamins/pills? YES NO
10. Does the patient have a history of or currently suffer from any of the following?
- Coronary Artery Disease YES NO
 - Heart failure YES NO
 - Valve (mitral or aortic) repair YES NO
 - Neurological disease (such as Parkinson's dementia or development delay) YES NO
 - Atrial fibrillation YES NO
 - GI Surgery YES NO
 - Cerebrovascular accident/stroke YES NO
 - Ankylosing spondylitis YES NO
 - COPD (Chronical Obstructive Pulmonary Disease) YES NO
 - Hiatal hernia YES NO

18805 Cox Avenue, Suite 250, Saratoga, CA 95070 • p: 408-624-1488 • info@capsovision.com

The CapsoCam Plus video capsule system is intended for the visualization of the small bowel mucosa in adults. It may be used as a tool in the detection of abnormalities of the small bowel. © 2020 CapsoVision, Inc. All rights reserved. 05-28-2020