

# Capsule Cleaning Instructions<sup>1</sup>

Prior to downloading data from the used CapsoCam Plus® capsule, it must be cleaned.



Always handle capsules with gloves.

## 1 Clean the capsule using ENZOL® Enzymatic Cleaner or equivalent according to the manufacturer's instructions<sup>2</sup>.

**1 oz.**  
(2 TBSP)

**4 Liters**  
(1 gallon)

Add one ounce (2 tablespoons) of ENZOL® Enzymatic Detergent per gallon of water, or follow manufacturer's directions on equivalent brand of detergent.

Soak capsule in sufficient solution to submerge completely (at least 5 cc). Organic matter should be removed. A minimum soak of one minute is recommended.



If necessary, manually clean capsule, including dimple on end of the capsule, with a soft bristle brush.

Thoroughly rinse capsule to remove detergent. Capsule is now ready for high-level disinfection.

Discard detergent after each use or when visibly soiled. Flush used solution down the drain with water or in accordance with local regulations.

## 2 Disinfect the capsule using Revital-Ox™ Resert® High Level Disinfectant\* or equivalent and rinse thoroughly, according to the manufacturer's instructions<sup>3</sup>.

Place pre-cleaned, rinsed and dried capsule in 5 cc of undiluted disinfection solution.

Soak capsule in sufficient solution to submerge completely (at least 5 cc) for a minimum of 8 minutes at  $\geq 20^{\circ}\text{C}$  (68°F).



Rinse the capsule with water.

Discard disinfection solution after each use. Flush used solution with water or in accordance with local regulations.

## 3 Dry capsule completely.\*\*



\* Revital-Ox™ RESERT® High Level Disinfectant has demonstrated disinfection efficacy on the presence of organic soil contamination and microbiological burden during reuse and may be reused for up to 21 days. Please see manufacturer's instructions for limitations and indications related to reuse.

\*\* Capsules with remnant moisture may have difficulty downloading data. Therefore, please ensure that the capsule is completely dried before attempting download.

### Sources:

- 1 CapsoVision, Inc., "Instructions for Use, CapsoCam Plus", IFU-2796 US (English).
- 2 Ethicon US, LLC "Enzol Automatic Detergent: Experience the Power of Fast-Acting Clean", AD-50255-01-US\_D, Promotional Data Sheet, 2014.
- 3 Steris Corporation, "Steris Revital-Ox™ RESERT® High Level Disinfectant Technical Data Sheet", 330-300-4455 Data Sheet, January 1, 2016.